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AeroGuard ‚Jellyfish‘

REF 2000.2000

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REF 2000.2000

Intended use

To protect patient and user from aerosoles that might originate during mechanical wound debridement.

Instructions

- 1 Remove AeroGuard from packaging and unfold (Note: do not perform unfolding directly above the wound area)
- 2 Insert handpiece of the mechanical debridement device (Note: do not perform insertion of handpiece directly above the wound area)
- 3 Make sure the absorbent sheet of the AeroGuard seals the working space properly (Recommendation: use an additional absorbent sheet underneath the patient's extremity)
 - Should you detect leakage of aerosoles, reposition or re-seal the working space with the AeroGuard, or replace the AeroGuard
 - If there might be a risk of contamination, please discard the AeroGuard according to the facility's regulations
 - Use of the AeroGuard only by trained personell
 - For use, handling and disposal follow your internal guidelines for hygiene standards
 - Single use product – do **NOT** reuse
 - Read instruction for use of mechanical debridement device carefully, in particular safety instructions and warnings

Warning

- Please avoid direct contact of the AeroGuard with the wound

Anwendung Application